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MCD-NU-002 Rév. : 8

Date de publication : 20Sep2021

NeoDiamond®
(Single-use sterile diamond-coated dental burs)
Instructions for use

PLEASE NOTE:

Product supplied only on medical prescription. These instructions, in whole or in part, are not a substitute for specialised training in diamond dental drills. Appropriate professional training is **MANDATORY** before using this device in clinical practice. NeoDiamond burs are intended for use by qualified dentists in dental clinics, hospitals, laboratories or schools for dental applications.

DESCRIPTION:

Microcopy's NeoDiamond burs are made from a single piece of high-quality stainless steel, coated at the operating end with natural diamond grit. The diamond burs are also with a unique protective coating, sometimes gold-plated. The range comprises models designed to meet the needs of all surgical and laboratory applications. The burs are packaged in a plastic bag in a clean room. Finally, they are sterilised by gamma rays.  Diamond burs are inserted in a dental handpiece which, by rotating, allows the user to cut or polish dental materials.

INDICATIONS

NeoDiamond burs are ideal for professionals who need to cut or polish dental materials. NeoDiamond diamond burs are designed to cut or polish a wide range of materials encountered in dentistry. This includes dental components such as enamel, dentin and bone, as well as dental materials such as amalgams, composites, glass-ionomer cements, polymers and precious and non-precious alloys.

CONTRAINDICATIONS

The use of Microcopy NeoDiamond burs is contraindicated in patients who are allergic to any of the components of Microcopy NeoDiamond product.

CLINICAL WARNINGS :

- a) Microcopy NeoDiamond burs are for single use only. Attempting to reprocess or re-sterilise them will impair their function.  
- b) **Do NOT** use the product if the packaging is open or damaged. 
- c) Do **NOT** use products that have passed their expiry date.
- d) Do **NOT** use if diamond grit or shanks are damaged.
- e) Do **NOT** force the burr, as this may break it and cause injury to the patient.
- f) Do **NOT** exceed maximum speed as this may cause unwanted overheating.
- g) Always keep the batch numbers of NeoDiamond strawberries to ensure traceability.

Failure to follow these instructions may result in the diamond burr becoming dull, broken or contaminated, as well as causing the following problems: infection, damage to the preparation field, injury to the patient or practitioner or potential aspiration or ingestion of the diamond burrs.

CLINICAL USE, PRECAUTIONS AND WARNINGS :

- a) Protect the patient's eyes and vulnerable tissues when using these diamond burs.
- b) Practitioners must wear protective goggles and a mask when using diamond burs.
- c) Surgical masks must be worn to prevent inhalation of aerosols and/or dust produced during the operation.
- d) Read the labelling on the packaging carefully to make sure you are using the correct device. Failure to do so may result in a delay in treatment or injury to the patient or practitioner.
- e) Follow the handpiece manufacturer's instructions for use and properly maintain and overhaul all handpieces.

- f) Make sure that the handpieces remain in good working order and are always properly lubricated so that the device is fully effective. Failure to maintain the handpieces correctly could result in treatment delays or injury to the patient or practitioner, aspiration or ingestion of the device, or damage to the preparation field due to the vibration of a worn chuck or turbine.
- g) Handle the packaging and product with care to ensure the integrity of the sterile barrier.
- h) If the packaging is damaged or has been unintentionally opened before use, the milling cutter must not be used and must be immediately discarded.
- i) For aseptic preparation, open the sachet just before use to prevent contamination.
- j) To open, hold the burr in the bag by the functional end, push the end of the shank through the plastic into the hand-tightening ring and tighten before releasing the grip on the functional end and discarding the empty bag.
- k) Ensure that the burr is fully inserted and securely fastened in the manual clamping ring before use. Failure to do so may result in the device 'popping out' of the handpiece and causing injury to the patient or practitioner, or aspiration or ingestion of the device.
- l) Never use force to insert a burr into the handpiece, as this could damage the manual clamping ring, which could delay treatment.
- m) When using short cutters (up to 20 mm long), make sure that the head/diamonds do not come into contact with the clamping mechanism. We recommend that you use short cutters with a mini handpiece.
- n) Before using the cutter, inspect it for broken and/or damaged kernels. potentially defective. Do not use worn or blunt devices.
- o) Dispose of any damaged diamond cutter immediately.
- p) Before use, run the handpiece to check for any faults, such as overheating.
- q) Do not exert excessive pressure on the cutter as this could cause unwanted overheating and/or failure. strawberry.
- r) Always move the drill when using it to avoid local overheating and/or damage to the drill. Unwanted heat generation can cause patient discomfort, tooth or tissue necrosis or burns.
- s) Avoid withdrawing the cutter at too steep an angle to prevent leverage and breakage, which could injure the operator. patient or practitioner.
- t) When cutting zirconia crowns or using for endo access on zirconia, apply a strong jet of water (>25ml/min) and use a light touch for pressure. Failure to do so may generate unwanted heat that could damage the dental pulp.
- u) Never exceed the maximum speeds indicated in the table below as this could cause undesirable overheating.

Head diameter the instrument 01/10 (mm) - ISO	Maximum authorised speed (rpm)	Recommended operating speed (rpm)
007 - 010	450 000	100 000 - 220 000
011 - 014	450 000	70 000 - 220 000
015 - 018	450 000	55 000 - 160 000
019 - 023	300 000	40 000 - 120 000
024 - 027	160 000	35 000 - 110 000
028 - 031	140 000	30 000 - 95 000
032 - 040	120 000	25 000 - 75 000
041 - 054	95 000	15 000 - 60 000
055 - 070	60 000	12 000 - 40 000
080 - 100	45 000	10 000 - 20 000

STORAGE (BEFORE USE) :

- Store strawberries in a dry, clean environment **at room temperature**. As a general rule, protect instruments from chemicals, acids, heat and severe temperature fluctuations.

- Poor storage conditions will reduce the shelf life of the products and may impair their functionality.

DISPOSAL :

- Used diamond burs should be disposed of in a biohazard sharps bin.
- Unused diamond burs can be disposed of in a conventional sharps waste bin.

SHELF LIFE OF STERILE PRODUCTS :

- Provided that all unopened sachets are properly stored and handled, the product will remain sterile for five (5) years, unless the sterile packaging is opened or damaged.
- Sterile products are labelled with their expiry date.

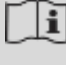
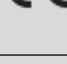
TRACEABILITY :

- Each pack is labelled with a **batch number** LOT
- This number must be quoted in all correspondence concerning the product.

WARNING: In the event of a serious incident involving the device, it must be reported to the manufacturer and, if appropriate, to the competent authority of the Member State in which the user and/or patient is located.

To request a free paper instruction manual, contact Microcopy at sales@microcopydental.com or 800.235.1863. The document will be sent to you within seven (7) days.

RELEVANT SYMBOLS :

	Manufacturer	Designates the manufacturer of the medical device.		Do not reuse	Refers to a medical device intended for a single use or for use on a single patient in a single operation. one-off intervention.
	Do not use if packaging is damaged	Designates a medical device that must not be used if the packaging has been damaged or open.		Do not re-sterilise	Designates a medical device that does not need to be re-sterilised.
	Batch code	Designates the manufacturer's batch code for the batch. to be identified.		Sterilised by irradiation	Designates a medical device which has been sterilised by irradiation.
	Deadline of use	Refers to the date after which the medical device should no longer be used.		Read the leaflet of use	Indicates to the user the need to consult the instructions use.
	CE marking	Designates the European conformity symbol		Authorised European Representative	Designates the authorised representative within the European Community.
	Resists humidity	Designates a medical device that must be protected from moisture.		DEVICE for professional use only	(ref US FDA CDRH) Designates a device to be used exclusively by a qualified professional.

	Medical devices	Indicates that the device is designed and intended for medical use.		Catalogue reference	Refers to the manufacturer's catalogue reference enabling the medical device to be identified.
	One patient, many uses	Refers to a medical device that can be used several times during the same life cycle. same operation.		Max. speed	Maximum speed
	Wear goggles	Indicates that protective goggles must be used.		Wear a mask	Indicates that a mask must be worn.
	Date of manufacture	Symbol for date of manufacture.		Importer	Designates the entity importing the medical device into the country recipient
	Distributor	Designates the entity distributing the medical device in the country recipient			

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